

USING THE DISPOSABLE NITROUSEAL® BREATHING CIRCUIT AND MASK WITH A NITROUS/OXYGEN FLOWMETER

Follow these easy instructions:

Step 1



Connect NITROUSEAL® Circuit to Mask.

Step 2



Connect NITROUSEAL® Circuit to Bag Tee on the Flowmeter.

Step 3



Connect the end of NITROUSEAL® Circuit exhalation limb to the nipple marked "MASK" on the vacuum module.

Step 4



Make sure: (1) the accordion-style corrugated tubing is configured to make a smooth "J" as shown, and (2) the collar at the top of the green bag is rotated so there is no twist at the bottom of the bag.

Step 5



Adjust fresh gas flow rate to desired level.

Step 6



Apply the NITROUSEAL® Mask first by placing under the chin then resting it snugly over the nose and mouth. Make sure the sealing cushion is not wrinkled.

Step 7



Adjust nitrous concentration to desired level.

Step 8




Dispose of NITROUSEAL® Circuit and Mask after each single patient use.

USING THE DISPOSABLE NITROUSEAL® BREATHING CIRCUIT AND MASK WITH A NITROUS/OXYGEN FLOWMETER

Follow these easy instructions:




Step 1: Connect NITROUSEAL Circuit to NITROUSEAL Mask.

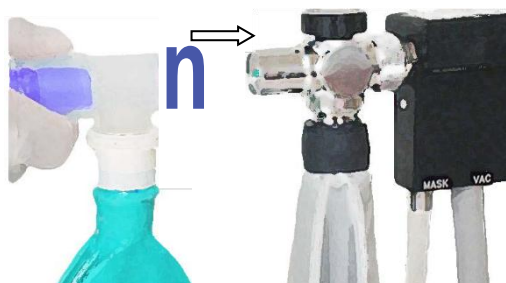

Step 2: Connect NITROUSEAL Circuit to Bag Tee on the Flowmeter unit.



Step 3: Connect the end of the NITROUSEAL Circuit exhalation limb to the nipple marked "MASK" underneath AVS module.

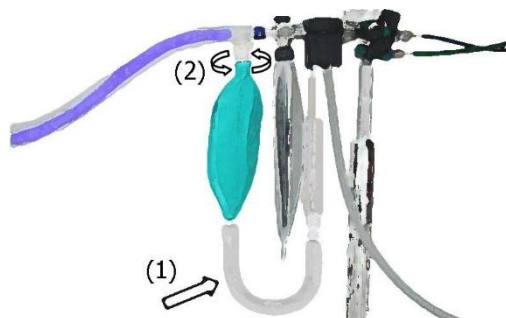

Step 4: Make sure: (1) the accordion-style corrugated tubing is configured to make a smooth "U" as shown, and (2) the collar at the top of the green bag is rotated so there is no twist at the bottom of the bag.



Step 5: Adjust O2 (oxygen) flow rate to desired level.



Step 6: Apply the NITROUSEAL Mask first by placing under the chin then resting it snugly over the nose and mouth. Make sure the sealing cushion is not wrinkled.



Step 7: Adjust N2O (nitrous) flow rate to desired level.



Step 8: Dispose of NITROUSEAL Circuit and NITROUSEAL Mask after each single use.

SEDATION
SYSTEMS

150 Pinellas Bayway S
Suite #208
Tierra Verde, FL 33715

www.sedationsystems.com • Tel: 888-959-5288

One or more of the following U.S. Patents may be applicable to the Nitrouseal® mask and/or breathing circuit: 8,336,549; 8,826,905; 10,722,674; © Sedation Systems, LLC 2024. All rights reserved.

A-IFU SedSys MX
Rev. H